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# Perfectly Engineered

## Feel drier and more confident with TENA Men

TENA Men is specifically designed for male bladder weakness. TENA Men provides long lasting protection, leaving you to get on with your day.

Try TENA Men for yourself with a FREE sample pack

Call 0845 3080 803 quoting - TENAMPE charged at UK standard local rate

Visit online [www.tenamen.co.uk](http://www.tenamen.co.uk)

Text TENAMPE your full name, First line of your address and postcode to 62277 (standard network rates apply)

Sample requests are limited to one per person in a 6 month period

# Pro dámy

Tena Her  
Designs

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## TENA *Pants* For independence

TENA *Pants* has all the premium features to provide increased independence with total comfort and security.

TENA *Pants* has added benefits for:

### Active and mobile

- Underwear-style design
- Breathable, soft and quiet material
- Odour protection for confidence

### Dementia

- Easy pull up design with tear away sides
- Underwear look and feel
- Highly absorbent with FeelDry technology\*

### Disability and/or restricted mobility

- Body-Close Fit for a more secure feeling
- Soft stretch elastics
- Leakage barriers and double leg cuffs for added protection\*

Open the TENA Men packaging and unfold the pad, removing the tab to expose the adhesive strip.

Make sure that the wide pad is positioned up

- 1 Body-Close Fit for a more secure feeling
- 2 FeelDry for excellent absorption\*
- 3 Soft breathable material for skin integrity
- 4 Underwear-like for easy use and to promote toileting

### Fitting guide

1. Pull up like underwear, ensure leg elastics are fitted well into the groin.

2. To remove – pull down like underwear or use the easy tear-away side seams.

\*Available on TENA Pants Normal, Plus and Super.