





SAMPelvicSling

Pánevní pás


SAM Pelvic Sling™ II instructions

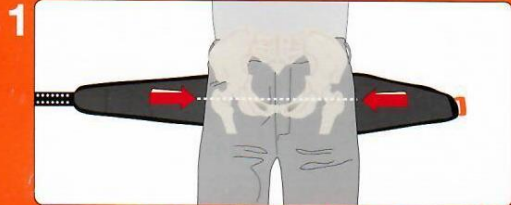

1 Remove objects from patient's pockets or pelvic area. Place SAM Pelvic Sling™ II black side up beneath patient at level of trochanters (hips).

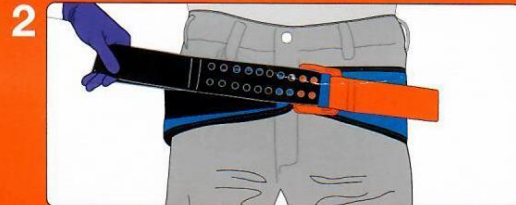

2 Place **BLACK STRAP** through buckle and pull completely through.

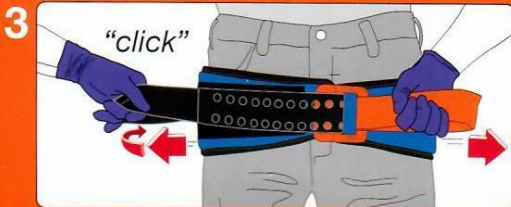

3 Hold **ORANGE STRAP** and pull **BLACK STRAP** in opposite direction until you hear and feel the buckle click. Maintain tension and immediately press **BLACK STRAP** onto surface of SAM Pelvic Sling™ II to secure.

Note: do not be concerned if you hear a second "click" after SAM Pelvic Sling™ II is secured.

TO REMOVE, lift **BLACK STRAP** by pulling upward. Maintain tension and slowly allow SAM Pelvic Sling™ II to loosen.

⚠ The SAM Pelvic Sling™ II should remain firmly attached to the patient while in the MRI room.

In case of prolonged use, monitor patient's skin.



Emergo Europe
Molenstraat 15
2513 BH-The Hague, The Netherlands

Sli-203-Civ-Back-2


WITH PATENTED AUTOSTOP BUCKLE

SAM Pelvic Sling™ II

FOR STABILIZATION OF PELVIC FRACTURES
WITH THE CORRECT FORCE

NO CUTTING, NO MEASURING, NO GUESSWORK

TRAINING VIDEO

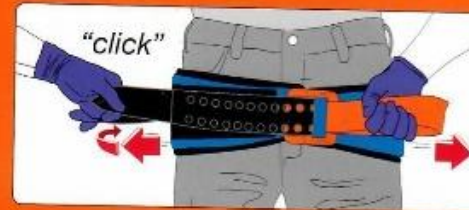

Placement:
The correct
level of
application is
at the greater
trochanters.



ONE PIECE DESIGN
APPLIES IN
3 EASY STEPS

LIMITS COMPRESSION TO A SAFE AND EFFECTIVE FORCE

SAM MEDICAL PRODUCTS

27350 SW 95th Ave, Ste 3038
Wilsonville, OR 97070 USA
800.818.4726 | sammedical.com
customerservice@sammedical.com

SAM Medical Products
Meets ASTM Standard F2428-04
ISO 13485 Certified
Made in USA
US Patent #8,554,784
SU-203-Civ-530-1



STANDARD
hip circ. 32-50" / 81-127 cm

VIDEO



VIDEO

⚠ CAUTION ⚠

THE SAM[®] PELVIC SLING MUST NOT BE APPLIED
OVER SOLID OBJECTS WITHIN THE CLOTHING.